Conversation Clear Plus

ConC 400

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This product contains two rechargeable batteries. If you think the product / a coin rechargeable battery has been swallowed or is lodged in the body, seek medical assistance immediately.

Avoid prolonged skin contact with the product, rechargeable battery, charger or charging cable when connected to a power supply. Components can heat up during charging and cause skin irritation.

Dispose of defective products with built-in rechargeable batteries at special collection points or return them to your specialist dealer to facilitate recycling.

Intended use / Liability
This product is intended to improve concert perception in noisy environments, to enable audio communication such as music playback and phone calls from TV, audio videos, tablets, smartphones, etc., and to improve a live speaker's ability to appear in the audience's field of vision. The intended use of the charging case is to charge the earbuds.

It is considered improper use when the product is used for any application not named in the corresponding product documentation. Sonova Consumer Hearing GmbH is not liable for damages that cannot be covered by the respective specifications (USB). Sonova Consumer Hearing GmbH is not liable for damages resulting from the loss of connection due to exceeding the Bluetooth transmission range. Sonova Consumer Hearing GmbH is not liable for damages arising from misuse or improper use of this product and its attachments/accessories.

Before putting the product into operation, please observe the respective country-specific regulations.

Notes on data collection and processing and on firmware updates
This product stores individual settings such as the volume and the Bluetooth address of paired devices. This data is required for the operation of the product and is transferred to Sonova Consumer Hearing GmbH or its companies commissioned by Sonova Consumer Hearing GmbH and is not processed.

Using the "Senneiser Conversation Clear" app, you can update the firmware of the product of free charge via an internet connection. If the device with which the app is installed is not connected to the internet, the update is automatically transferred to and processed by Sonova Consumer Hearing servers in order to provide and transfer suitable firmware updates: hardware identifier, hardware revision number, firmware version, operating system type (Android, iOS) and version, location information. The data is not used for the purpose of providing and transferring the firmware updates and is not stored permanently. If you do not want this data to be collected, please contact us at service@senneiser-hearing.com.

Manufacturer Declarations
For more information on regulatory compliance markings, refer to the supplied support sheet.

Warranty
Sonova Consumer Hearing GmbH offers a warranty of 24 months on this product. For more information on the warranty, please visit our website www.senneiser-hearing.com/warranty or contact your Sonova Consumer Hearing partner.

USA AND NEW ZEALAND ONLY
Sonova Hearing Australia Pty Ltd provides a warranty of 24 months on these products. For the current warranty conditions, visit the Sonova Consumer Hearing website: www.senneiser-hearing.com/warranty

EU Declaration of conformity
This product is in compliance with the requirements that cannot be excluded under Australian and New Zealand Commonwealth law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. Your warranty is unaffected by any other legal provisions that may impose a higher standard of acceptable quality and the failure does not amount to a major failure. This warranty is in addition to other rights or remedies under law. Nothing in this warranty excludes, limits or modifies any remedy available to the consumer which is claimed by law.

To make a claim under this contract, trace a case via Sonova Consumer Hearing website: www.senneiser-hearing.com/warranty or contact your local dealer. All expenses of claiming the warranty will be borne by the person making the claim. Sonova Consumer Hearing international warranty is provided by: Sonova Hearing Australia Pty Ltd (ABN 8256194229) The Zenth, Level 14, Tower A, 821 Pacific Highway, Sydney, NSW 1585, Australia.

In compliance with the following requirements
• General Product Directive (2001/95/EC)
• Compliant to Sound Pressure Limits according to country specific requirements.

UK Declaration of conformity
This product is in compliance with the requirements that cannot be excluded under Herby, Sonova Consumer Hearing GmbH declares that the radio equipment type ConCPlus1 (ConCPlus1-R, ConCPlus1-L, ConCPlus1-C) is in compliance with the EMC Directive 2014/53/EU and the Low Voltage Directive 2014/35/EU. The full text of the EU declaration of conformity is available at the following Internet address: www.senneiser-hearing.com/download.

UK Declaration of conformity
This product is in compliance with the requirements that cannot be excluded under the Radio Equipment Regulations (2017).

Notes on disposal
• EU: Battery Directive (2006/66/EC & 2013/56/EU)
• EU: WEEE Directive (2012/19/EU)
• UK: Battery Regulations (2013)
• UK: WEEE Regulations (2013)
The symbol of crossed-out wheeled bin on the product, the battery/rechargeable battery (if applicable) and/or the packaging indicates that these products must not be disposed of with normal household waste, but must be disposed of separately at the end of their operational lifetime. For packaging disposal, refer to the legal regulations on waste segregation applicable in your country. Improper disposal of packaging materials can harm your health and the environment.

The separate collection of waste electrical and electronic equipment, batteries/rechargeable batteries (if applicable) and packagings is used to promote the reuse and recycling and to prevent negative effects on your health and the environment. Do not dispose of the contents of the product, especially of used batteries, in the household electrical and electronic equipment and batteries/rechargeable batteries at the end of their operational lifetime in order to make contained recyclable materials available for reuse. If batteries/rechargeable batteries can be removed without damaging them, you are obliged to dispose of them separately (for the safe removal of batteries/rechargeable batteries see the instruction manual). Handled separately from other waste. Batteries/rechargeable batteries with special marks, as they pose particular risks, such as the risk of fire and/or the risk of ingestion in the case of coin batteries. Reduce the generation of battery waste as much as possible by using longer-life batteries or rechargeable batteries.

Further information on the recycling of these products can be obtained from your municipal administration, from the municipal collection points, or from your Sonova Consumer Hearing partner. You can also return electrical or electronic equipment to distributors who have a take-back obligation. Hereby, you make an important contribution to the protection of the environment and public health.

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